

## Realize Language Quick Reference Guide for PRC Accent devices with Empower




### The Realize Language (RL) Service

The Realize Language service is a web-based product that helps you see how a client is using a PRC communication device. Data is collected on the device, which can then be uploaded to the Realize Language site and analyzed. Realize Language converts data into a number of easy-to-use reports that make understanding how someone is using words much easier.

### STEP 1: CREATE A PRC ACCOUNT

To set up a Realize Language account, you must create a free *Everything PRC!* account. The email and password you use for your *Everything PRC!* account will be the same that you use to access the Realize Language website. If you already have one of these, then you can skip this section.

1. Go to <https://account.prentrom.com> and select **Create Account**.

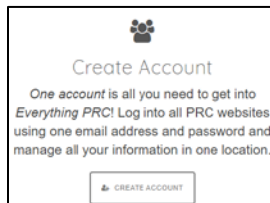

2. Enter your details into the various boxes on the **Create Account** page. The email and password that you use are also the ones you will use for your Realize Language account.
3. You have now created your *Everything PRC!* account and can use the email/password for your Realize account.

### STEP 2: ACTIVATE YOUR RL ACCOUNT

Your Accent device includes a FREE one-year subscription to the Realize Language website. Your subscription is activated using a special CODE that is on a postcard in the shipping box. If you purchase a subscription online, the code is sent to you in an email.

1. Log in to your *Everything PRC!* account using <https://account.prentrom.com>
2. On the **Account Overview** page, in the right-hand corner, you will see “Have an Activation Code?” with a box into which you can **enter your activation code**.
3. Select **Check Code** and you should see a screen that confirms your Realize Language account. If you get a “code not valid” message, check your code and try again.
4. Your Realize Language account is now active.

### STEP 3: LOG INTO YOUR RL ACCOUNT


1. Go to <https://realizelanguage.com>
2. Click on the **Login** link at the top right of the screen.




3. Enter your *Everything PRC!* email and password, then select **Login**.
4. The first time you log into your Realize account you will be on a page called **Manage People** and will be asked to **Create a New Person**.

### STEP 4: CREATE A NEW PERSON IN YOUR RL ACCOUNT

1. In the box headed Name, enter a name or identifier for the person whose data you will be tracking.

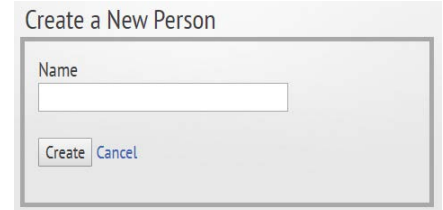

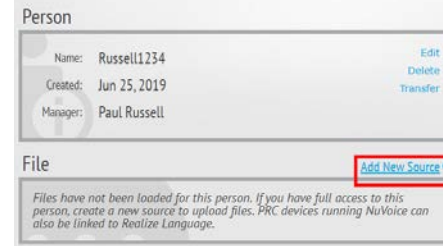2. On the new page that opens, click on the link **Add New Source**. You must now provide a name for the device on which data is being tracked. [Russell1234 » Summary](#)
- 
3. Use the Name box to give the device a name and, if you want, add any other comments in the Description box. You don't have to include a description but you must have a name.
  4. Select the **Dashboard** tab on the top and you will go to a page called **Overview** for the person you are tracking.
  5. You have now set up a Person in your account and are ready to upload data from your device.

### STEP 5: TURN DATA LOGGING ON

When a new device is shipped, the data logging feature is turned OFF by default. You have to turn data logging ON.

1. Select **Menu** button at the top right.




2. Select **Toolbox** button.



3. Select the **Vocabulary** tile.
4. Select the **Data Logging** tile.


5. Set the slider next to Data Logging to On.

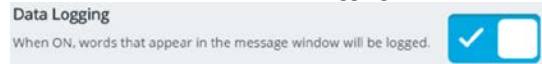

6. Select **Close**, then **Exit**. Your device will now log what is said and when.

#### STEP 6: SAVE A DATA LOG FILE TO A USB FLASH DRIVE.

1. Plug a USB flash drive into the device.
2. Select **Menu**, and then **Toolbox**.
3. Select the **Vocabulary** tile.
4. Select the **Data Logging** tile.
5. Select **Save Data Log to File**.
6. Select the button labeled **Select Folder**.
7. A window will open showing your USB drive, and your data file is automatically given a name along with a date and time.

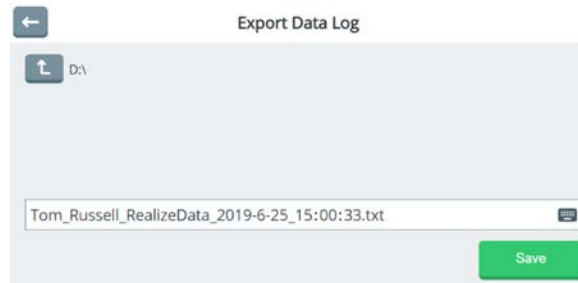

8. Select the **Save** button.
9. After the message “Export Succeeded” appears briefly, select the **Close** button then **Exit**.

Your file is now saved to your USB flash drive and now you can go to your Realize Account to upload it.

#### STEP 7: UPLOAD A DATA LOG FILE TO THE REALIZE LANGUAGE WEBSITE.

1. Go to <https://realizelanguage.com> and login.
2. Go to your **Dashboard** and select the **Manage** tab.
3. Select **File** on the top right.
4. Select **Select a file to Upload**.
5. Using the window that pops up, find the file you just saved on your flash drive and select it.
6. Click **Open** and your file name will now appear where Select a file to Upload was.
7. Click on **Upload**.

Your file is now uploaded and you can go to your Dashboard to start looking at the data.